

GET IT STRAIGHT. REDUCE THE WAIT.

LABELS MUST BE INTACT AND READABLE

PROPERLY LABELED



Make sure initials (if added) do not interfere with bar code



- Place label directly under cap
- Name at the top of the tube
- Barcode straight along the length of the tube
- Place label on top of manufacturer label
- Leave visible window to see blood

DOUBLE CHECK ALL LABELS BEFORE SENDING

Required Patient Info → Name, DOB, Date of Draw

- Place only one label on each tube
- Send all extra labels in the pocket of the specimen bag

UNACCEPTABLE



Wrong Orientation

Upside Down

Crooked

Wrong Orientation



Scrunched

Bunched

Wrinkled